



ACCESSING THE SKINBUTTON FOR FLUID DELIVERY

Accessing the SkinButton for fluid delivery is a simple procedure and can be performed by the owner at home. Each accessing needle (18 Gauge, 3/8" blunt) is provided in a sterile pouch together with an alcohol cleansing pad to ensure compliance with aseptic accessing technique. All Client Needle Kits contain accessing suggestions.

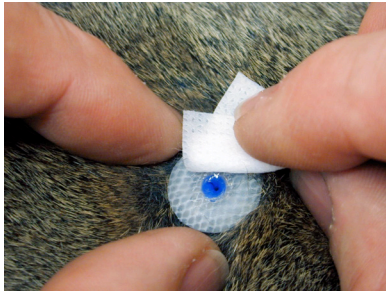

1) Using accepted aseptic technique, wipe the SkinButton & the surrounding area with the cleansing pads provided, or a suitable disinfectant before proceeding with the therapy. After wiping, the area should be left to air dry for 3-4 minutes to allow the disinfectant to work. (DO NOT blow on the area).

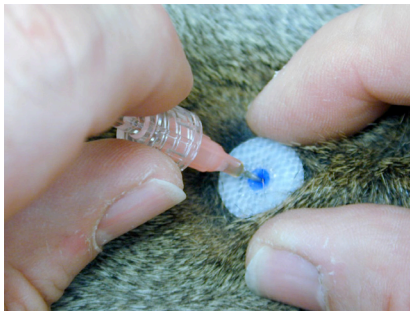

2) Remove the accessing needle from the pouch, leaving the needle protector in place. Attach the needle to the fluid source - syringe or infusion bag.

3) Remove the needle protector and stabilize the perimeter of the SkinButton before inserting the needle into the blue septum in the center of the SkinButton.



4) Deliver the required amount of fluid.

5) Once the required amount of fluid is delivered and the treatment complete, stabilize the SkinButton as you remove the accessing needle. Dispose of the needle in an appropriate manner.

manufactured by



847-674-7143

info@norfolkvetproducts.com